




Overcome the challenges of male urethral dilation.

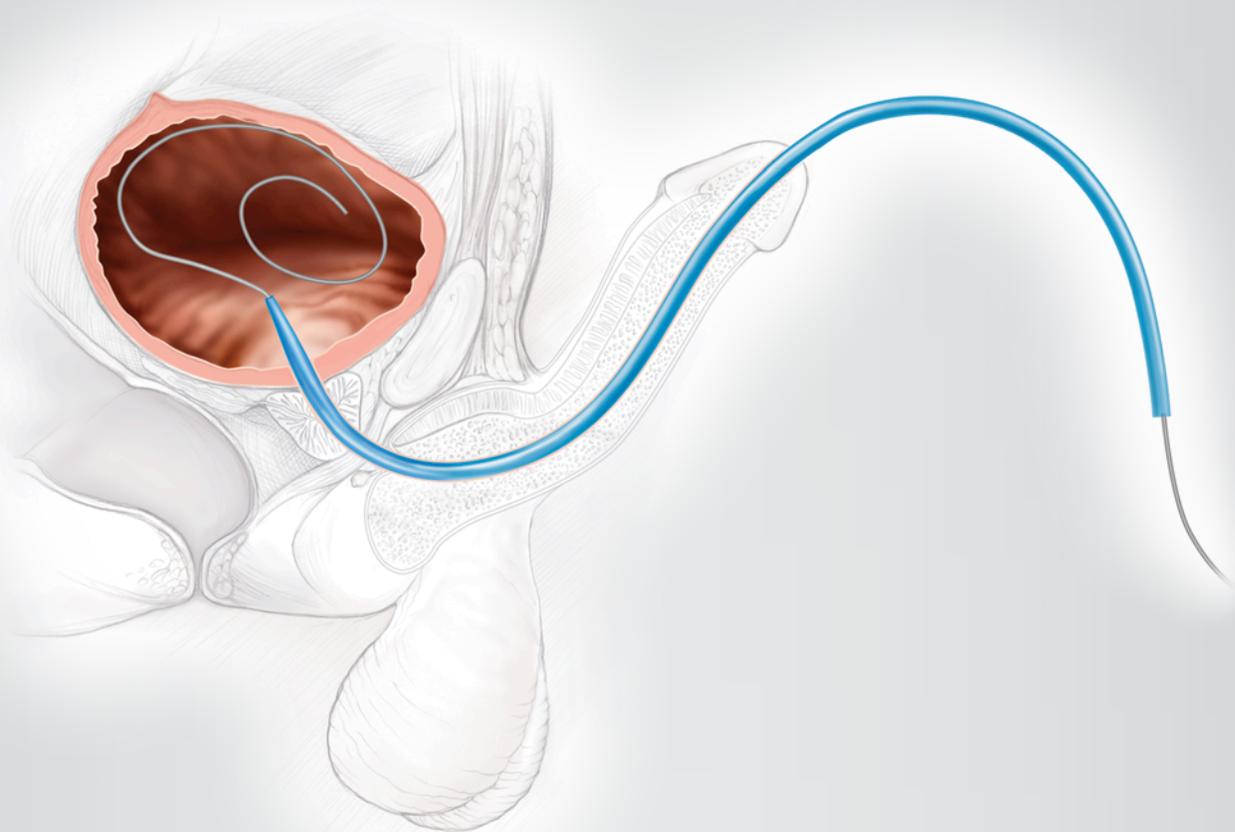

S~Curve™
URETHRAL DILATOR SET



www.cookmedical.com

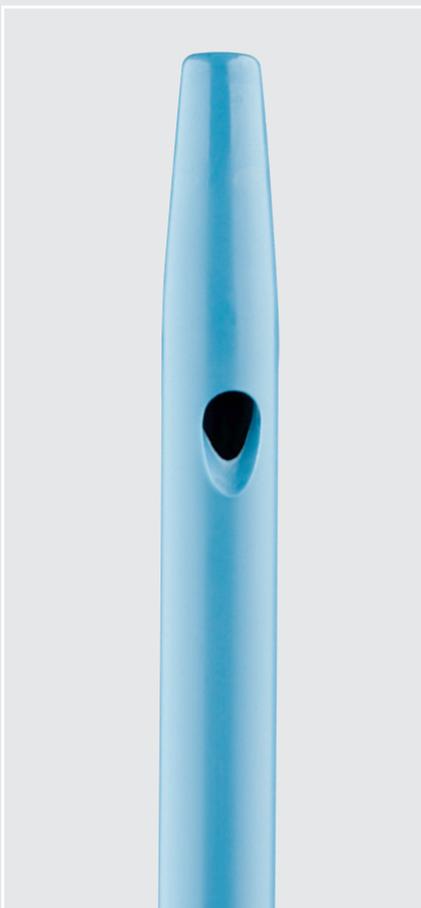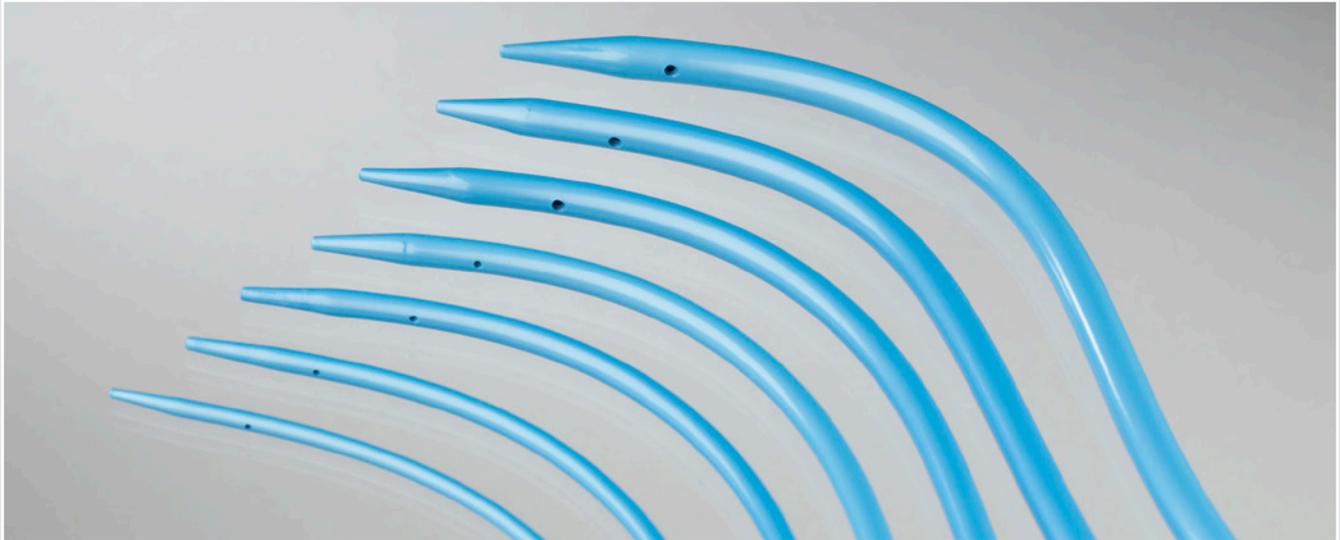

S~Curve™ URETHRAL DILATOR SET

Use the S~Curve™ Urethral Dilators for the dilation of male urethral strictures and vesical neck contractures. Each S~Curve dilator is contoured to ease passage and reduce trauma by supporting the natural curvature of the male urethra.


Features and Benefits

S~Curve Dilators

- The AQ® hydrophilic coating creates a low-friction surface for effortless dilator advancement across tight strictures.
- The lumen allows for precise advancement over a previously positioned wire guide.
- The ported tip allows for urine drainage to confirm proper placement.


Convenient Tray Design

- The reservoir allows for the hydration and activation of the AQ® hydrophilic coating while the dilator is still in the tray.
- The embossed sizes allow for quick identification and size selection.

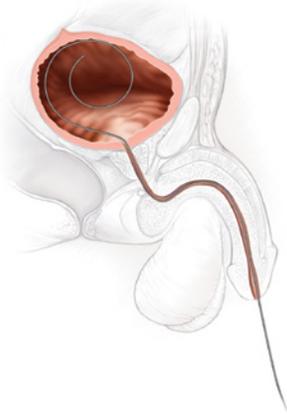

STEP 1

Under direct vision or with fluoroscopic control, position a wire guide through the urethra and curl the wire guide into the bladder. Consider using a HiWire® hydrophilic nitinol wire guide for difficult anatomy or if a stricture is present. After you establish access you can exchange the HiWire for a stiff-bodied wire guide.


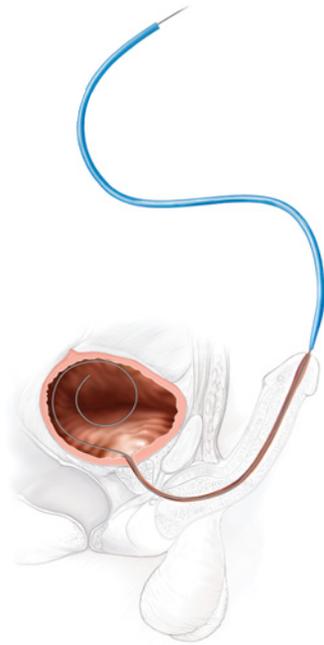


STEP 2

Hold the penis in an upward position. Introduce the dilator and advance it over the previously placed wire guide.


ALTERNATIVE METHOD

Hold the penis in the downward position. Introduce the dilator over the previously placed wire guide. Advance the dilator to the level of the bulbar urethra. With the dilator positioned at the level of the bulbar urethra, rotate the dilator 180 degrees.

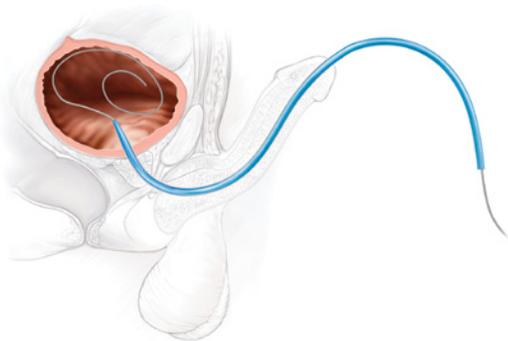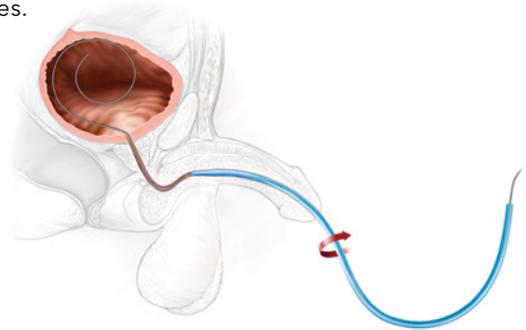

STEP 3

Continue to advance the dilator into the bladder. Urine drainage will confirm proper placement.

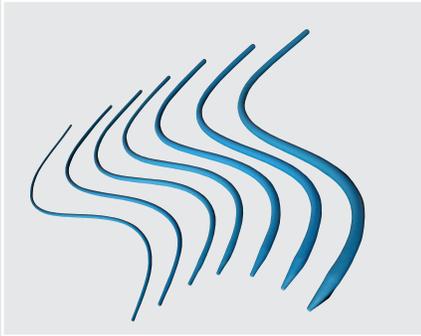

S~Curve™

URETHRAL DILATOR SET

Used for dilation of male urethral strictures and vesical neck contractures.

Order Number	Reference Part Number	Fr	Length cm
Set With Sideports			
G32789	073701-CD	8.0-20.0	37
Dilator Only With Sideports			
G46249	073722-CD	22.0	37
G46250	073724-CD	24.0	37
G46251	073726-CD	26.0	37

Some products or part numbers may not be available in all markets. Contact your local Cook representative or Customer Service for details.

Customer Service - Urology

EMEA: EDI - www.cookmedical.com/edi.do
 Distributors: +353 61239240, ssc.distributors@cookmedical.com
 Austria: +43 179567121, oe.orders@cookmedical.com
 Belgium: +32 27001633, be.orders@cookmedical.com
 Denmark: +45 38487607, da.orders@cookmedical.com
 Finland: +358 972519996, fi.orders@cookmedical.com
 France: +33 171230269, fr.orders@cookmedical.com
 Germany: +49 6950072804, de.orders@cookmedical.com
 Hungary: +36 17779199, hu.orders@cookmedical.com
 Ireland: +353 61239252, ie.orders@cookmedical.com
 Italy: +39 0269682853, it.orders@cookmedical.com
 Netherlands: +31 202013367, nl.orders@cookmedical.com
 Norway: +47 23162968, no.orders@cookmedical.com
 Poland: +48 223060159, pl.orders@cookmedical.com
 Spain: +34 912702691, es.orders@cookmedical.com
 Sweden: +46 858769468, se.orders@cookmedical.com
 Switzerland - French: +41 448009609, fr.orders@cookmedical.com
 Switzerland - Italian: +41 448009609, it.orders@cookmedical.com
 Switzerland - German: +41 448009609, de.orders@cookmedical.com
 United Kingdom: +44 2073654183, uk.orders@cookmedical.com
www.cookmedical.com

Americas: EDI - www.cookmedical.com/edi.do
 Phone: +1 812.339.2235, 800.457.4500, Fax: 800.554.8335
 E-mail: orders@cookmedical.com

Australia:
 Phone: +61 734346000, 1800777222, Fax: +61 734346001, 1800077283
 E-mail: cau.custserv@cookmedical.com

